

# **Brucella Test Kit Introduction:**

The Bovine Brucella Antibody Rapid Detection Test Kit is designed to detect Brucella infection in cattle. The kit is convenient and easy to use by the farmer, veterinarian or LSS. Brucella testing is rapid and completely cost-effective. Which helps in early detection and control of diseases.

#### Features:



Accurately identifies Brucella virus infections and their species in animals.



Brucella test kit testing is best and cost effective as compared to conventional methods.



Brucella testing is done in a minimal blood sample.



The test does not require a lab or any other equipment.

#### Advantages :



The test shows results in just 10-15 minutes



Useful for protecting animal health



Less financial investment

storage

No need for cold




Easy to use portable kit



@ info@lkvetcare.in



Help prevent financial loss

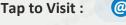

Optimized in the second sec

1 of 2

# Intended Use of Brucella Test Kit:

Bovine Brucella abortus antibody rapid test is a tool used to detect Brucella antibodies in cattle. It
works using a method called immunochromatography. in which whole blood, plasma or serum
samples from cattle are tested. by which it is determined whether animals have Brucella
antibodies or not.

### Summary and Test Description:

- Bovine brucellosis is a disease that causes abortion in cattle. This bacterium can also infect cows, buffaloes. Some animals can act as transmitters of bacteria. Some animals can act as transmitters of the bacteria, making it difficult to control the disease in cattle.
- Bovine Brucella Rapid Test is designed to detect antibodies. In which a method called colloidal gold conjugated detection antibody is used. Detection of antibodies in cattle helps in early detection of Brucella abortus infection. So that appropriate measures can be taken to control the disease and prevent its spread.
- Through contact with infected animals, Brucella abortus can infect humans. Also, consumption of contaminated dairy products also leads to infection, which can affect various organs of humans.

# Specifications and Usage:

Parasites detected	IGG Antibodies to Brucella abortus, melitensis	Sample type	Whole blood, serum, plasma
shelf life	24 months	Storage	2-30°C
Capture antigen	Purified lps antigen	Sensitivity	96.80%
Antigen specificity	99.10%	Comparison of the test	Complement determination test
Species	Bovine, Ovine and Caprine	Packing	10 Test Bulk Uncut Sheets

# How to take a sample for testing:





Leave 10 $\mu l$  of sample in the cassette





10 min. Wait (for buffer-2 drops)

 Test Interpretation:

 Image: Contract of the second secon

### **Contents of Test Kit:**

Foil Pouch (10-Nos) 1) Test Cassette 2) Dropper 3) Desiccant 4) Assay Diluent Dropper Bottle 5) Instruction Sheet



For Purchase of Brucella Test Kit **Contact:** LK Vet Care Pvt. Ltd., 3/A, Rachana Avenue, FC Road, Deccan Gymkhana, Pune - 9552524949 **Also available at: Pandharpur** Animal Lab: Wakratund Building, Near KBP College Chowk, Wakhri Road, Pandharpur - 8956210651